

# Controller 2.0

Smart  
Reliable  
Accurate

## Settings Screen Guide

Your VAD Team may ask you for information from your Controller 2.0. Here is a step-by-step guide.



## To enter settings screens



From the home screen, press and hold the scroll button for two seconds. During a low or medium priority alarm you may return to the home screen by pressing the scroll button.

**NOTE:** Setting screens are not accessible during a high priority alarm.

## To exit settings screens




Press and hold the scroll button for two seconds.


**NOTE:** The system will automatically exit the controller setting screens after 60 seconds of inactivity or if an alarm sounds.

## Below are the available screens. To scroll through them, press the scroll button on your controller.


### Battery Cycles

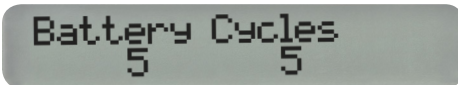


### Low Flow Alarm Limit

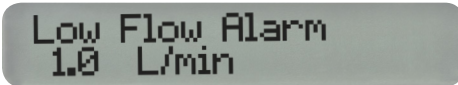


### High Power Alarm Limit

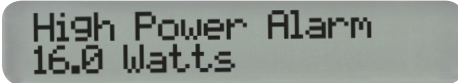


### Hematocrit

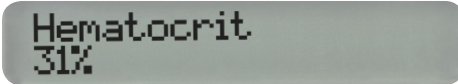

### VAD Speed

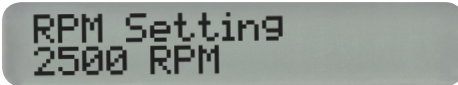


### Suction Response

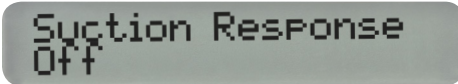

### Lavare™

(this screen will only appear if Lavare Cycle is enabled)

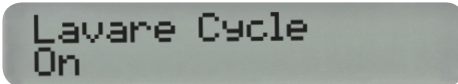

### VAD ID



### Peak




### Trough




### Implant Date



### Patient ID

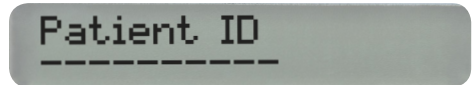

### Software Version

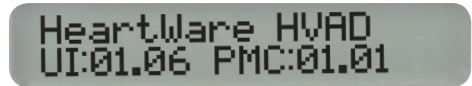

**HeartWare®**

HeartWare, Inc.  
14400 NW 60th Avenue  
Miami Lakes, FL 33014 USA  
[www.heartware.com](http://www.heartware.com)

HEARTWARE, HVAD, Lavare and the HEARTWARE logo are trademarks of HeartWare, Inc.

©2016 HeartWare, Inc. INTL1082 Rev01 08/16  0086